

earClin®

3in1 ear spray

1. WHY IS IT ADVISED TO CLEAN OUR EARS?
2. WHAT IS EARCLIN 3IN1 EAR SPRAY?
3. HOW SHOULD EARCLIN 3IN1 EAR SPRAY BE USED?
4. CAN EARCLIN 3IN1 EAR SPRAY BE USED BY PEOPLE WITH A HEARING AID?
5. ARE THERE RISKS ASSOCIATED WITH THE USE OF EARCLIN 3IN1 EAR SPRAY?
6. CONTENT
7. CONTACT DETAILS

GENERAL ADVICE:

Read this user manual carefully before you start using earClin 3in1 ear spray, even if you have used this medical device before. The information could have been adapted according to the latest insights and data when you start a new package. Keep this manual, it may be necessary to read it again.

1. WHY IS IT ADVISED TO CLEAN OUR EARS?

Earwax (cerumen) is produced naturally by the body. It protects the skin of the ear canal, assists in cleaning and lubrication, and provides protection against bacteria, fungi, and water. The wax usually works itself out through chewing and other jaw movements. Some people produce more earwax than others, and those who use hearing aids, earplugs and in-ear headphones can suffer from increased ear wax build-up. The shape of the ear canal can also complicate the natural removal of wax. When the earwax cannot escape properly, the ear can get clogged, and a clogged ear can affect the hearing and/or balance.

The carefully designed and developed products of earClin can prevent the need for manual removal of earwax and are an excellent alternative for the use of cotton buds. These can potentially damage the eardrum and even worsen the problem by pushing excess earwax deeper into the ear canal.

2. WHAT IS EARCLIN 3IN1 EAR SPRAY?

EarClin 3in1 ear spray treats the most common ear complaints. Use earClin 3in1 ear spray to:

1. Relieve pain & irritation
2. Treat earwax plugs
3. Dissolve accumulated ear wax

The 100% natural EarClin 3in1 ear spray is composed of herbal oils making it suitable for both adults and children aged 6 months and older. It has been specially developed to address ear pain, irritation, earplugs, and accumulated ear wax.

3. HOW SHOULD EARCLIN 3IN1 EAR SPRAY BE USED?

1-2 sprays in the ear, up to 3 times a day. Shake well before use. EarClin 3in1 is a medical device, do not use earClin 3in1 ear spray for more than 30 consecutive days.

Method of application: When the pack is used for the first time, it is advisable to shake briefly before use. Have a tissue at hand. Bring the nozzle close to the ear canal. Spray 1-2 times. Press the ear canal shut and massage the ear for 5-10 seconds. Wipe the auricle clean. It is essential to clean the spray head thoroughly after use for hygienic reasons. Clean thoroughly with warm water and dry with a tissue.

4. CAN EARCLIN 3IN EAR SPRAY BE USED BY PEOPLE WITH A HEARING AID?

earClin 3in1 ear spray is suitable for people who wear a hearing aid. Regular use (2-3 times a week) ensures optimum hygiene of the ear canal, which contributes positively to the functioning of the hearing aid.

5. ARE THERE RISKS ASSOCIATED WITH THE USE OF EARCLIN 3IN1 EAR SPRAY?

There are no known adverse side effects from the ingredients used in earClin 3in1 ear spray.

Allergies

Stop using the device, and consult a general physician if there is any allergic reaction or oversensitivity to one of the ingredients.

USE DURING PREGNANCY AND / OR BREASTFEEDING:

EarClin 3in1 ear spray can be used with confidence during pregnancy or breastfeeding. If in doubt, consult your treating physician first.

COMBINATION WITH MEDICINES:

It is not recommended to use earClin 3in1 ear spray if you are already using medicines in the ear.

USE IN CHILDREN:

Because earClin 3in1 ear spray consists of 100% plant based oils and ingredients, it can be used on children as young as 3 months. When doing so, do not forget to pay attention to what is written under "Warnings".

⚠ WARNING:

Do not apply the ear spray in cases of eardrum perforation or eardrum tubes. Do not spray into the eyes. If the spray is accidentally into the mouth or eye, rinse immediately with plenty of water. Keep out of the reach of young children.

6. CONTENT

- earClin 3in1 • Instruction for use

INGREDIENTS: OLEUM EUROPEAE, SAW PALMETTO EXTRACT, PHYTOSTEROL COMPLEX, BISABOLOL, CINNAMOMUM ZEYLANICUM (CINNAMON) LEAF OIL, SESAMUM INDICUM (SESAME) OIL, SEA WHIP EXTRACT, MURRAYA KOENIGII (CURRY) LEAF OIL, EUGENIA CARYOPHYLLUS (CLOVE) LEAF OIL, SPEARMINT OIL. FREE FROM PRESERVATIVES

METHOD OF STORAGE:

This product should be stored in a cool place, away from sunlight at room temperature and not exposed to temperatures above 50°C. Keep out of reach of young children.

📅 EXPIRATION DATE:

Do not use after expiry date: see bottom of pack

EXPLANATION OF SYMBOLS

The warning signs and symbols on the packaging and instructions for use are essential to ensure that you use this product safely and correctly and to protect you and others from injury. The symbols are explained in the table below:

Symbol	Explanation	Symbol	Explanation
	Symbol for the manufacturer		Symbol that indicates the Unique device identifier
	Symbol that indicates that instructions should be consulted for use		Importer
	Symbol that indicates that caution is necessary when operating the device		Keep out of reach of children
	EU Authorized Representative		Symbol for the Lot number
	Upper limit of temperature (50°C)		Symbol that indicates that the item is a medical device
	Waste disposal		Use by: (expiry date)
	Not for children aged 3 months and younger		

Any serious incident that has occurred in relation to this product should be reported to the manufacturer and to the competent authority of the Member State in which the user is established.

7. CONTACT DETAILS



Distributor:

TheOTClab
Fred. Roeskestraat 115,
1076 EE Amsterdam
The Netherlands
www.theotclab.com



Manufacturer:

Naveh-Pharma LTD.
11-19, Yad Harutzim St., P.O. 8139
Netanya, 4250519
Israel
www.navehpharma.com



OBELIS S.A.

53 Boulevard
Général Wahis 1030
Brussels,
Belgium

15ml     