

earClin[®]

earshower

FOR KIDS!

1. WHY DO YOU NEED TO CLEAN YOUR EARS?
2. WHAT IS EARCLIN EARSHOWER FOR KIDS?
3. HOW TO USE EARCLIN EARSHOWER FOR KIDS
4. CAN EARCLIN EARSHOWER FOR KIDS BE USED BY CHILDREN WHO WEAR A HEARING AID?
5. ARE THERE RISKS ASSOCIATED WITH THE USE OF EARCLIN EARSHOWER FOR KIDS?
6. WHY IS EARCLIN EARSHOWER FOR KIDS SAFE TO USE?
7. CONTENT
8. CONTACT DETAILS

1. WHY DO YOU NEED TO CLEAN YOUR EARS?

Earwax, or cerumen, is a self-cleaning agent that your body produces. It collects dirt, bacteria, and other debris. Usually, the earwax works its way out of the ears naturally through chewing and other jaw motions. Earwax can build up and affect your hearing. When earwax reaches this level, it's called impaction. If you have impaction, you may experience symptoms like:

- Aching in the affected ear
- Fullness or ringing in the ear
- Impaired hearing in the affected ear
- An odor coming from the affected ear
- Dizziness
- A cough

You may be more likely to develop excess earwax if you use hearing aids, ear plugs or in-ear headphones. Older adults and people with developmental disabilities have a bigger chance of developing excess earwax. Your ear canal's shape may make the natural removal of wax difficult.

2. WHAT IS EARCLIN EARSHOWER FOR KIDS?

EarClin earshower is developed based on the same technique used by ear specialists. EarClin earshower contains a unique soap which simply flush out excess earwax. It gently rinses your ear canal using water enriched with the unique EarClin soap solution. A simple method for cleaning ears EarClin earshower is an effective solution for regularly cleaning the ear canals in the comfort and convenience of your own home. Like brushing your teeth, it should be part of your personal health and hygiene routine. Use the soapy solution in EarClin earshower to simply flush out excess earwax.

EarClin earshower for Kids is particularly helpful for:

- Children with regular exposure to water
- Children with heavy production of cerumen
- Children who wear hearing aids, ear protectors, or in-ear earphones
- Children with regular exposure to dust or dirt

3. HOW TO USE EARCLIN EARSHOWER FOR KIDS?

Use EarClin earshower on a regular basis (2-3 times a week) to keep the outer ear canal clean. In case of excessive ear wax (cerumen), use EarClin earshower every day for a duration of 2 consecutive weeks. Use EarClin earshower for a maximum of 30 days after first use.

EarClin earshower for Kids is indicated for:

- Children applied by an adult
- Children with hearing aids
- Children who want to keep their outer ear canal clean
- Children (3+ years and older), applied by an adult

The best time to use EarClin earshower is during your usual shower or bath routine. First time use: flush out EarClin earshower 2 times with clear water, after that suck lukewarm water (max 40°C) into the device, flush the ear, then dry. Like the professional cleaning method used by specialists, EarClin earshower cleans the ears with natural, lukewarm water. The soap stick (medical soap) inside the tube not only facilitates the cleaning process but also indicates how much longer the EarClin earshower device can be used for.

How to use EarClin earshower for kids?

Suction bulb: Use the suction bulb to draw lukewarm water (max 40°C) up into the EarClin earshower and to gently release it into the ear canal. Then use the empty EarClin earshower to blow air into the ear canal, removing any remaining moisture.

Squeeze to fill the EarClin earshower once with lukewarm water. Rinse the ear 3 times per ear at every use. The suction bulb does not need to be refilled in the meantime.

Spray nozzle: Place the spray nozzle at the entrance of the ear canal. It's shape and size stops you from accidentally pushing the EarClin earshower too far into your ear.

Gentle and effective soap: When mixed with lukewarm water, the ingredients contained in the EarClin earshower produces an unique soap solution.

4. CAN EARCLIN EARSHOWER FOR KIDS BE USED BY CHILDREN WHO WEAR A HEARING AID?

EarClin earshower is particularly intended for people who wear a hearing aid for its regular use (use each day before going to bed) allows the perfect hygiene of the ear canal which is indispensable to the proper operation of the device.

5. ARE THERE RISKS ASSOCIATED WITH THE USE OF EARCLIN EARSHOWER FOR KIDS?

EarClin earshower is well tolerated and is specifically developed for all people, above the age of 3. Keep away from children below 3 years. Small

parts present a choking hazard - keep out of reach of children. Stop the use of EarClin earshower if there is any allergic reaction. Do not use if you are allergic to one of the ingredients.

⚠ Do not use without medical opinion in case of otitis, pain in the ear, a perforated ear drum and generally, by any person subject to dizziness or with medical antecedents relating to the ear or ear drum. Without knowing it, you may have a cerumen impaction and it is possible that under the action of EarClin earshower this impaction swells causing discomfort. In this case, consult a physician.

EarClin earshower must not be applied

- In persons with pre-existing damage (e.g., perforation) or deformation of the ear drum
- In persons with pre-damaged ear canal
- In children under 3 years of age

⚠ If your ear canals are blocked, we recommend visiting an ENT specialist. After the ears have been professionally cleaned by a specialist we recommend flushing them regularly with the EarClin earshower to prevent further earwax build-up.

6. WHY IS EARCLIN EARSHOWER FOR KIDS SAFE TO USE?

EarClin earshower thoroughly flushes out the cerumen without any mechanical intervention. The rounded shape of the ear piece makes it feel pleasant against the user's ear. The dimensions prevent it from entering too far into the ear canal, making injuries virtually impossible, especially to the ear drum. At the same time, the ear piece acts as a spray head. Its patented design ensures that the flushing solution is optimally dispersed within the ear canal. The olive-shaped ear piece also features a mechanism to protect against excessive pressure. Keep your EarClin earshower within easy reach when taking a shower or bath.

7. CONTENT

- 1 EarClin earshower • Instruction for use

INGREDIENTS: SODIUM TALLOWATE, GLYCERIN, AQUA (WATER), SORBITOL, SUCROSE, PEG-4001, TETRASODIUM.

EXPLANATION OF SYMBOLS

The warning signs and symbols on the packaging and instructions for use are essential to ensure that you use this product safely and correctly and to protect you and others from injury. The symbols are explained in the table below:

Symbol	Explanation	Symbol	Explanation
	Symbol for the manufacturer		Symbol that indicates the Unique device Identifier
	Symbol that indicates to consult the instructions for use		Not for children aged 3 years and younger
	Symbol that indicates that caution is necessary when operating the device		Keep out of reach of children
	Date of manufacturing		Symbol for the Lot number
	Indicates a medical device that needs to be protected from moisture		Symbol that indicates that the item is a medical device

Any serious incident that has occurred in relation to this product should be reported to the manufacturer and to the competent authority of the Member State in which the user is established.

9. CONTACT DETAILS

Distributor

TheOTClab
Fred. Roeskestraat 115
1076 EE Amsterdam
The Netherlands
www.theotclab.com

Manufacturer

TheOTClab
Fred. Roeskestraat 115
1076 EE Amsterdam
The Netherlands
www.theotclab.com

